<b>DE</b>	<b>Folienkassette</b>	<b>SV</b>	<b>Kassetter</b>
<b>GB</b>	<b>Foil cassette</b>	<b>DA</b>	<b>Foliekassette</b>
<b>FR</b>	<b>Support de Plaque</b>	<b>FI</b>	<b>Levykasetit</b>
<b>ES</b>	<b>Casete de laminillas o placas</b>	<b>NO</b>	<b>Foliekassett</b>
<b>IT</b>	<b>Caricatori</b>	<b>RU</b>	<b>Кассеты для рентгеновской плёнки</b>
<b>NL</b>	<b>Foliecassettes</b>	<b>EL</b>	<b>Κασέτες φύλλων</b>
<b>PT</b>	<b>Cassete de película</b>	<b>PL</b>	<b>Kasety na płyty obrazowe</b>

**Bestellinformationen / Ordering Information**

<b>Size 0</b>	<b>2x3</b>	1x	2130-010-00
<b>Size 1</b>	<b>2x4</b>	1x	2130-011-00
<b>Size 2</b>	<b>3x4</b>	1x	2130-012-00
<b>Size 3</b>	<b>2,7x5,4</b>	1x	2130-013-00
<b>Size 4</b>	<b>5,7x7,6</b>	1x	2130-014-00
<b>OPG</b>	<b>12,7x29</b>	<b>5x12 Inch</b>	1x 2130-021-00
<b>OPG</b>	<b>15x30</b>		1x 2130-022-00
<b>Ceph</b>	<b>18x24</b>		1x 2130-023-00
<b>Ceph</b>	<b>24x30</b>		1x 2130-024-00
<b>Ceph</b>	<b>20x24</b>	<b>8x10 Inch</b>	1x 2130-025-00
<b>Ceph</b>	<b>13x18</b>		1x 2130-026-00

2009/10/13

DÜRR DENTAL AG  
Höpfheimer Strasse 17  
74321 Bietigheim-Bissingen / Germany  
Fon: +49 7142 705-0  
www.duerr.de / info@duerr.de

**ES Casete de laminillas o placas**

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**GB Foil cassette**

The foil cassette is designed to ease the introduction of the image plate into the VistaScan. It also serves to hinder the accidental deletion of X-ray information on the image plate through stray ambient light.

**Do not autoclave the foil cassette**

Refer also to the Installation and Operating Instructions for the Dürr VistaScan.

**Intraoral foil cassette, size 0 - 4 with tongue (not all model types)**

The tongue on the lever is designed to support the sliding of the image plate from the foil cassette into the VistaScan.

**Do not remove this tongue (fig. 1)!**

- Slide the lever until it engages at the lower catch on the foil cassette. (Fig. 1)
- Lay the light protection cover onto the tongue. Slide the image plate out of the light protection cover into the foil cassette until it engages the upper catch. (Fig. 2)

The active side of the image plate must lie completely on the tongue of the foil cassette. (Fig. 3)

**Foil cassettes for OPG and Ceph formats**

- Remove the foil cassette from the X-ray film cassette and place in position in the upper tube of the VistaScan. (Fig. 4, step 1)
- Now slide the foil cassette downwards onto the transport arc and position correctly. (Fig. 4, step 2)

The foil cassette is correctly in position when the foam rubber lies flush with the entry slot of the transport arc. (Fig. 5)

- Slide the image plate through the opening out of the foil cassette and into the VistaScan until it is automatically engaged. (Fig. 6)

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